

## Tutor courtesy card – Roger Touchscreen Mic/Roger X

The Roger Touchscreen Mic/Roger X radio system can be a great help to a hearing aid user – a little of your time to help understand what does what will be much appreciated.



The tutor wears the Roger Touchscreen Mic to pick up their voice, the student wears a Roger X radio receiver to feed sound directly into their hearing aid.

The result is a greatly improved and relaxed listening experience for the hearing aid user.



There is no need to be self conscious in wearing a radio microphone – all the best TV stars do so.

### 5 information points

**1** The student should give you the Roger Touchscreen Mic transmitter turned on and ready to use – if you can see the display then the unit is on.




**2** The Touchscreen Mic should be worn about the length of a biro from the mouth.



**3** Avoid wearing 'noisy' clinking jewellery.



**4** During the lesson the Roger Touchscreen Mic will automatically adjust to louder voice levels so you need not change the way you normally talk.

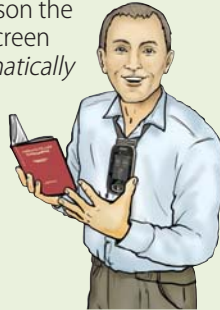

**5** During the lesson try not to draw attention to the Roger X user but do help them by always repeating questions from the floor.

**6** Be aware that you have a direct sound link to Roger X users in the room and mute the microphone when you are talking to others.

## Tutor courtesy card *continued*

## Roger Touchscreen Mic/Roger X

### Roger Touchscreen Mic controls

In normal use, it should not be necessary to use the controls – we explain them here in case you need to use them!

#### Microphone mute status

When not addressing the radio aid system user, always mute the microphone with a short press of the mic mute button (don't forget to toggle back to normal afterwards!).

When the mic is muted, red light will flash.



#### Charging and multimedia connection



The battery status is shown in the display and very low battery is shown by a double flashing red light.



The Roger Touchscreen Mic needs daily charging.

Connection facilities into AV equipment are available using the lead supplied, please contact us for more information.



After the session, if you are not returning the transmitter to the user, remember to turn it off before going to the staff room or toilet! The Roger Touchscreen Mic has a potential transmitting range of several classrooms.



### Thank you for your time



### Connevens Limited

Bridge House, 1 Nutfield Road, Merstham, Surrey RH1 3EB

Helpline 01737 247571

Information: [www.connevens.info](http://www.connevens.info)

info@connevens.com

Online shopping: [www.DeafEquipment.co.uk](http://www.DeafEquipment.co.uk)